## DiaSpecT Hemogloblin Cuvettes REF 90C.0001 EN

The DiaSpecT Tm System consists of the DiaSpecT Tm analyzer and the DiaSpecT Hemogloblin Cuvettes. Only use the DiaSpecT Tm analyzer with the DiaSpecT Hemogloblin Cuvettes. Please read the User Manual for proper use of the system.

**Intended Purpose**  
DiaSpecT Hemogloblin Cuvettes are intended for use with the DiaSpecT Tm analyzer for the determination of hemoglobin in capillary and venous whole blood (K, EDTA or lithium heparin). For near patient testing by healthcare professionals for laboratory testing by laboratory professionals.

**DiaSpecT Hemogloblin Cuvette**  
**Packaging content:** DiaSpecT Hemogloblin Cuvettes packed in re-closable bags of 100 pieces, 5 bags per box, package insert (this document).

**Storage and Handling of the Cuvettes**  
The cuvettes are made of plastic (PMMA) and contain no active agents. The cuvettes are for in-vitro diagnostic use only. The Cuvettes require 10 µL sample volume and serve both as pipette and measuring cuvette.

**Storage and Handling of the Cuvettes**  
Store at 0 °C to +50 °C, Temperatures of -30 °C to +70 °C are temporarily permitted (± 24 h) when stored in the original package. Use the Cuvettes prior to the expiry date. Unused Cuvettes should be stored in the original bag. Do not touch the optical eye of the cuvette. Do not touch the cuvette with hands most with disinfectants as this could impact the cuvette material.

**Performance Characteristics**  
**Precision**  
Precision was tested in accordance with CLSI guideline EP05-A3 showing results summarized below.

	High Sample CV (%)	Mid sample CV (%)	Low Sample SD (µg/L)
repeatability	0.5	0.8	0.05
reproducibility	0.6	0.8	0.06
in-to-to-precision	0.2	0.6	0.05

**Accuracy**  
The trueness was determined by direct method comparison against the international reference method HCN.

	N	Min [g/L]	Max [g/L]	Slope (95th percentile)	Correlation coefficient (r)	Relative deviation (Hb range 50 - 200 µg/L)
Analyzer 1	22	1	255	0.978 (0.971-0.985)	1.000	(-1.37) - (-2.34) %
Analyzer 2	22	1	255	0.979 (0.971-0.987)	1.000	(-1.34) - (-2.34) %

The results of the comparison studies between the DiaSpecT Tm and a comparative device performed at the point-of-care are summarized in the following table. The study was performed across four external sites.

Sample Type	N	Min	Max	Slope (Passing-Bablok regression)	Kendalls Tau	Bland-Altman (95th percentile)
K,EDTA	355	4.1 g/dL	24.5 g/dL	0.938	0.844	0.2 (-1.2 to 6.6)
L-Heparin	120	10.4 g/dL	20.0 g/dL	1.000	0.890	0.5 (-1.4 to 1.3.2)
Capilar	368	8.5 g/dL	20.1 g/dL	1.000	0.723	-0.2 (-1.14 to 11.1)

**Non-interfering Substances**  
The following substances do not affect the test results (highest tested concentrations in brackets): conjugated Bilirubin (37 mg/dL), unconjugated Bilirubin (37 mg/dL), total Cholesterol (482 mg/dL), Intralipid (2366 mg/dL), Hemolysis free hemoglobin (100% / 18.9 mg/dL), creatinine (16 mg/dL), Urea (129 mg/dL), Uric Acid (29 mg/dL), Acetaminophen (19 mg/dL), Acetylaceticacid (4 mg/dL), Ascorbic Acid (6 mg/dL), Dopamine (0.1 mg/dL), Iupropfen (26 mg/dL), Tetracycline (2.6 mg/dL), 3x K, EDTA (19 mg/dL), 3x Li-Heparin (Tubo filled 1/3 volume), 3x Li-Heparin (Tubo filled 1/3 volume), Leucocytes (11 x 10<sup>9</sup>/L), Thrombocytes (794 x 10<sup>9</sup>/L), Microcytes (79.1 fL), Tricyclerides (11.1 mm), CO-Hemoglobin (7.2 %).

**Non-interfering Conditions**  
The following conditions do not affect the test results (highest tested concentrations in brackets): conjugated Bilirubin (37 mg/dL), unconjugated Bilirubin (37 mg/dL), total Cholesterol (482 mg/dL), Intralipid (2366 mg/dL), Hemolysis free hemoglobin (100% / 18.9 mg/dL), creatinine (16 mg/dL), Urea (129 mg/dL), Uric Acid (29 mg/dL), Acetaminophen (19 mg/dL), Acetylaceticacid (4 mg/dL), Ascorbic Acid (6 mg/dL), Dopamine (0.1 mg/dL), Iupropfen (26 mg/dL), Tetracycline (2.6 mg/dL), 3x K, EDTA (19 mg/dL), 3x Li-Heparin (Tubo filled 1/3 volume), 3x Li-Heparin (Tubo filled 1/3 volume), Leucocytes (11 x 10<sup>9</sup>/L), Thrombocytes (794 x 10<sup>9</sup>/L), Microcytes (79.1 fL), Tricyclerides (11.1 mm), Carboxyhemoglobin (7.2 %).

**Interfering Conditions**  
The following conditions may interfere with the test results in malformation of red blood cells (e.g. sickle cell disease, Thalassemia) can lead to false high results.

**Limitations**  
The optimal calculated hematocrit value is displayed for information only and should not be used for clinical decisions.

**References**  
NCCLS/CLSI document H15-A3, Reference and Selected Procedures for the Quantitative Determination of Hemoglobin in Blood; Approved Standard-Third Edition (2010), Dacie and Lewis, Practical Haematology, Twelfth Edition, Elsevier Limited 2017

**Manufacturer**  
DiaSpecT Medical GmbH  
Ebenforder Chaussee 3, 39179 Barleben, Germany  
E-mail: support@ekf-diagnostic.de

**CE Mark**  
The product is CE marked. The CE mark indicates that the product conforms to the requirements of the applicable European Union directives.

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