

S-Sedivette®


User Guide

1



To collect blood using the aspiration method, attach S-Sedivette® to a Safety-Needle immediately before collection.

2

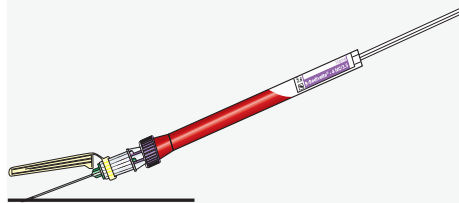


Proceed with blood collection in accordance with your internal instructions.

3




After blood collection, lock plunger into S-Sedivette® base and break off piston rod!

4

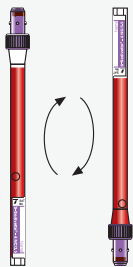

Mix sample thoroughly immediately after blood collection by inverting the S-Sedivette® 5 times upside down. Make sure that the air bubble/glass bead always moves from one end to the other. Do not shake rapidly.

5

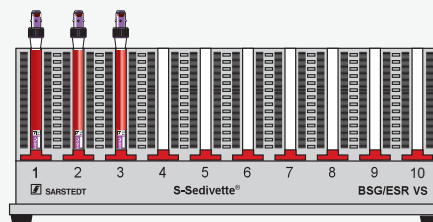


ESR determination should be carried out within a period of 4 hrs. after blood collection. Mix the S-Sedivette® thoroughly prior to ESR determination once again and place it into the S-Sedivette® rack. Use the knurled thumb screw (arrow) to adjust each sample to zero.

Barcode labelling instructions:




Important: Attach label at S-Sedivette® bottom.

Correct !

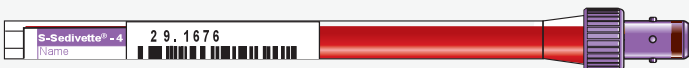

Wrong !

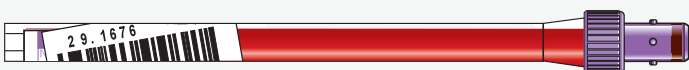

Wrong !

SARSTEDT AG & Co.
P.O. Box 12 20
D-51582 Nümbrecht
Phone +49 2293 305-0
Fax +49 2293 305-122
info@sarstedt.com
www.sarstedt.com

 SARSTEDT