Instructions for Use

CellPath

A STATLAB COMPANY

Eosin

PUTTS EOSIN

RBB-0100-00A

RBB-0100-00B

RBB-0100-00C

RBB-0100-00D

EOSIN 1% (ALCOHOLIC)

RBC-0201-00A

RBC-0201-00B

RBC-0201-00C

RBC-0201-00D

EOSIN 1% (AQUEOUS)

RBC-0100-00A

CellPath Ltd

80 Mochdre Enterprise Park
Newtown, Powys, SY16 4LE, UK

T: +44 (0)1686 611333

E: info@cellpath.com

cellpath.com

Putts Eosin Contains: Potassium Dichromate

Eosin 1% Alcoholic Contains:
Methanol

Emergo Europe,
Westervoortsedijk 60,
6827 AT Arnhem,
The Netherlands

Intended Use:

A chemical agent(s), dye(s) and/or solution(s) intended for the artificial colouration of specific target tissue structures, intra/extracellular elements and/or infectious microorganisms, in a clinical specimen, for their subsequent visualization, examination, identification and/or differentiation.

For Laboratory professional use

Storage and Stability:

Store in a dry, cool and well-ventilated place. Avoid exposure to direct sunlight.

Keep the container tightly closed and properly labelled when not in use.

Recommended storage temperature: 4-40°C.

Refer to the product label for the expiry date.

For additional safety and handling information, consult the SDS.

Product Appearance – **Pink**

Putts Eosin

A ready to use solution with no preparation required.

Eosin 1% Alcoholic and Aqueous

Acidification required before use:

- For a 325–350 ml bath: add 1 ml of 10% glacial acetic acid
- For a 1L bottle: add 3 ml of 10% glacial acetic acid

Alternatively, add 3–4 drops of 99% glacial acetic acid per 325–350 ml bath

Mix thoroughly and ensure the final pH is between 5.0 and 5.4

pH Considerations (All grades):

pH may rise during staining due to carryover from blueing reagents and water washes. Optimal eosin staining occurs at pH 5.0–5.4.

Recommendations to Maintain Optimal pH:

1. Pre-Eosin Step: Use an 80% IDA bath for 15–30 seconds before eosin staining to reduce water carryover

2. Post-Staining Wash:

- a. For Eosin Aqueous and Putts Eosin: Limit water wash to 15 seconds to prevent excessive eosin loss
- b. For Eosin Alcoholic: Use a 15-second wash in 80% IDA before dehydration, clearing, and mounting

3. Periodic Adjustment: Add 3–4 drops (approx. 0.5 ml) of glacial acetic acid to 325–350 ml of eosin to restore optimal pH.

NOTE: If pH is correct but staining is weak, the eosin solution is likely exhausted and should be replaced

Dispose of the reagents in line with government guidelines.

Please contact info@cellpath.com with any additional questions.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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