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SARSTEDT S-Monovette® Safety-Kanülen / Kanülen

DE	Gebrauchshinweis – SARSTEDT S-Monovette® Safety-Kanülen / Kanülen	1 - 4
EN	Instructions for use – SARSTEDT S-Monovette® safety needles / needles	5 - 8
BG	Инструкции за употреба – SARSTEDT S-Monovette® Safety Канюли / Канюли	9 - 12
CS	Návod k obsluze – SARSTEDT S-Monovette® jehly Safety / jehly	13 - 16
DA	Brugsanvisning – SARSTEDT S-Monovette® Safety-kanyler / kanyler	17 - 20
EL	Οδηγίες χρήσεως – SARSTEDT S-Monovette® Βελόνες ασφαλείας / βελόνες	21 - 24
ES	Instrucciones de uso – agujas y agujas de seguridad S-Monovette® SARSTEDT	25 - 28
ET	Kasutusjuhend – SARSTEDT süsteemi S-Monovette® Safety-kanüülid / kanüülid	29 - 32
FR	Mode d'emploi – Aiguilles de sécurité / Aiguilles SARSTEDT S-Monovette®	33 - 36
HR	Uputa za upotrebu – SARSTEDT S-Monovette® Sigurnosna Iгла	37 - 40
HU	Használati utasítás – SARSTEDT S-Monovette® Safety kanül / kanülök	41 - 44
IT	Istruzioni d'uso – SARSTEDT Aghi Safety S-Monovette® / aghi S-Monovette®	45 - 48
KO	사용 설명서 – SARSTEDT S-Monovette® 안전 바늘 / 바늘	49 - 52
LT	Naudojimo instrukcijos – SARSTEDT „S-Monovette®“ saugios kaniulės / kaniulės	53 - 56
LV	Lietošanas norādes – SARSTEDT S-Monovette® Safety adatas / adatas	57 - 60
NL	Gebruiksaanwijzing – SARSTEDT S-Monovette® Safety-naalden / naalden	61 - 64
NO	Bruksanvisning – SARSTEDT S-Monovette® Safety-kanyler / kanyler	65 - 68
PL	Instrukcja obsługi – Igly Safety / igły S-Monovette® SARSTEDT	69 - 72
PT	Instruções de utilização – Agulhas de segurança Safety / agulhas SARSTEDT S-Monovette®	73 - 76
RO	Instrucțiuni de utilizare – Ace cu protecție / ace SARSTEDT S-Monovette®	77 - 80
RU	Инструкция по применению – Иглы / безопасные иглы S-Monovette® SARSTEDT	81 - 84
SK	Návod na Použitie – SARSTEDT S-Monovette® bezpečnostná Safety ihla / ihla	85 - 88
SL	Navodila za uporabo – SARSTEDT S-Monovette® varnostne kanile / kanile	89 - 92
SV	Bruksanvisning – SARSTEDT S-Monovette® Safety-kanyler / kanyler	93 - 96
TH	คำแนะนำในการใช้งาน – แคนนูล้ำความปลอดภัย / แคนนูล้ำ S-Monovette® ของ SARSTEDT	97 - 100
TR	Kullanım için talimatlar – SARSTEDT S-Monovette® Safety kanülleri / kanüller	101 - 104
ZH	使用说明 – SARSTEDT S-Monovette® 安全针 / 针管	105 - 108

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Instructions for use

SARSTEDT S-Monovette® Safety needles/needles

DE	Gebrauchshinweis – SARSTEDT S-Monovette® Safety-Kanülen/Kanülen	1 - 4
EN	Instructions for use – SARSTEDT S-Monovette® safety needles/needles	5 - 8
BG	Инструкции за употреба – SARSTEDT S-Monovette® Safety Канюли/Канюли	9 - 12
CS	Návod k obsluze – SARSTEDT S-Monovette® jehly Safety/jehly	13 - 16
DA	Brugsanvisning – SARSTEDT S-Monovette® Safety-kanyler/kanyler	17 - 20
EL	Οδηγίες χρήσεως – SARSTEDT S-Monovette® Βελόνες ασφαλείας/βελόνες	21 - 24
ES	Instrucciones de uso – agujas y agujas de seguridad S-Monovette® SARSTEDT	25 - 28
ET	Kasutusjuhend – SARSTEDT süsteemi S-Monovette® Safety-kanüülid / kanüülid	29 - 32
FR	Mode d'emploi – Aiguilles de sécurité/Aiguilles SARSTEDT S-Monovette®	33 - 36
HR	Uputa za upotrebu – SARSTEDT S-Monovette® Sigurnosna Iгла	37 - 40
HU	Használati utasítás – SARSTEDT S-Monovette® Safety kanül / kanülok	41 - 44
IT	Istruzioni d'uso – SARSTEDT Aghi Safety S-Monovette®/aghi S-Monovette®	45 - 48
KO	사용 설명서 – SARSTEDT S-Monovette® 안전 바늘/바늘	49 - 52
LT	Naudojimo instrukcijos – SARSTEDT „S-Monovette®“ saugios kaniulės / kaniulės	53 - 56
LV	Lietošanas norādes – SARSTEDT S-Monovette® Safety adatas/adatas	57 - 60
NL	Gebruiksaanwijzing – SARSTEDT S-Monovette® Safety-naalden/naalden	61 - 64
NO	Bruksanvisning – SARSTEDT S-Monovette® Safety-kanyler/kanyler	65 - 68
PL	Instrukcja obsługi – Igły Safety/igły S-Monovette® SARSTEDT	69 - 72
PT	Instruções de utilização – Agulhas de segurança Safety/agulhas SARSTEDT S-Monovette®	73 - 76
RO	Instrucțiuni de utilizare – Ace cu protecție/ace SARSTEDT S-Monovette®	77 - 80
RU	Инструкция по применению – Иглы / безопасные иглы S-Monovette® SARSTEDT	81 - 84
SK	Návod na Použitie – SARSTEDT S-Monovette® bezpečnostná Safety ihla / ihla	85 - 88
SL	Navodila za uporabo – SARSTEDT S-Monovette® varnostne kanile/kanile	89 - 92
SV	Bruksanvisning – SARSTEDT S-Monovette® Safety-kanyler/kanyler	93 - 96
TH	คำแนะนำในการใช้งาน – แคนนูล้ำความปลอดภัย / แคนนูล้ำ S-Monovette® ของ SARSTEDT	97 - 100
TR	Kullanım için talimatlar – SARSTEDT S-Monovette® Safety kanülleri / kanüller	101 - 104
ZH	使用说明 – SARSTEDT S-Monovette® 安全针/针管	105 - 108

Intended use

The S-Monovette® safety needle/needle is a sterile needle developed for single use. The S-Monovette® safety needle/needle and the SARSTEDT S-Monovette® together form a system for venous blood collection. The needle guard of the safety needle is used immediately after collecting blood with the last S-Monovette® to cover the used needle in order to reduce potential accidental needle stick injuries. The products are to be used by trained qualified personnel.


Product description

The S-Monovette® safety needle/needle is a sterile, non-pyrogenic single-use product packaged individually in a blister pocket. S-Monovette® safety needles/needles are composed of a medical 1-inch (25 mm) or 1 ½-inch (38 mm) long disposable stainless steel needle with a triple bevel and an integrated needle holder. The products are available in needle diameters of 20, 21, 22 and 23 gauge (G). The needle guard of the safety needles serves to cover the needle after use.

Product variant:

S-Monovette® safety needle

S-Monovette® needle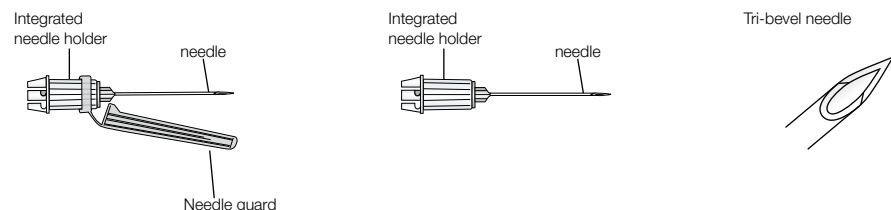

Safety information and warnings

1. Check all individual packaging for damage prior to use and, if damaged, do not use. Likewise do not use the product if the needle is bent or broken.
2. Follow the directions for handling.
3. Wear gloves and general personal protective equipment to protect yourself from blood and possible exposure to pathogens transmitted by blood.
4. Do not bend the needle.
5. During use and disposal, always keep your hands behind the needle.
6. After activating the safety mechanism of the S-Monovette® safety needle, it can no longer be opened. Do not use force.
7. When removing or disposing of it, note that blood may possibly leak out. It is therefore recommended to dispose of the S-Monovette® safety needle/needle with the tip of the needle facing upward with the (S-Monovette® safety needle) needle guard activated.
8. Dispose of used, contaminated S-Monovette® safety needles/needles immediately in suitable sharps disposal containers and also dispose of blood collection equipment (e.g. lancets, adapters) in sharps disposal containers.
9. Treat all biological specimens and sharp blood collection utensils (needles) according to the guidelines and procedures in your institution. In case of exposure to biological specimens or a needle prick injury, consult a doctor, as HBV, HCV, or HIV or other infectious diseases can be transmitted. Use needles with an integrated needle guard. The safety guidelines and procedures of your institution must always be followed.
10. Reusing this product could result in infections, injury or death.

Contraindications

1. Do not perform venous blood collection on patients with infections, burns, injuries or after vascular surgery at the planned puncture site.
2. Do not make injections of any kind with the S-Monovette® (safety) needle; it is to be used only for blood collection.

General residual risks of venous blood collection

When applying the basic technique for blood collection, there are the following general residual risks according to CLSI GP41 and the WHO guidelines on drawing blood:

1. Pain
2. Bruising
3. Vasovagal reactions
4. Infections
5. Nerve injuries
6. Allergic reactions, mainly caused by disinfectants.

Collecting and handling specimens

READ THIS DOCUMENT IN FULL BEFORE PROCEEDING WITH VENIPUNCTURE.

WORK MATERIALS REQUIRED FOR SAMPLING/BLOOD COLLECTION

1. All necessary S-Monovette® blood collection systems.
2. S-Monovette® safety needles/needles.
3. Gloves, gown, goggles or other suitable protective gear to protect against pathogens or potentially infectious materials transmitted by blood.
4. Labels for patient identification.
5. Disinfectant to clean the collection site (follow the guidelines of the facility for sterile specimen collection for preparing the collection site). Do not use alcohol-based cleaning agents if the samples are to be used for blood alcohol tests.
6. Dry, clean, disposable swabs.
7. Tourniquet.
8. Plaster or bandage.
9. Waste container for sharp/pointed objects for safely disposing of used materials.


Venipuncture with the S-Monovette® safety needles/needles

Apply tourniquet. Prepare the puncture site with a suitable disinfectant. Do not touch the venipuncture site after disinfection. Position the patient to allow easy access to the vein.


- 1 Open the blister packaging of the S-Monovette® safety needle/needle aseptically using the peel-off technique with your thumbs and index fingers **1a** or by tearing along and through the arrow **1b**. The transparent film must be facing upwards.
- 2 Put the S-Monovette® into the S-Monovette® safety needle/needle **(I)** and secure it with a **¼ clockwise rotation (II) (connect) 2a/2b**.
- 3 After the disinfectant has dried, remove the protective cover of the needle and ensure that the bevelled end of the S-Monovette® safety needle/needle is pointing upwards. Carry out venipuncture while grasping the S-Monovette® with the connected needle **3a**. Then collect blood with the S-Monovette® by grasping the integrated needle holder with the thumb and index finger of one hand and gently retracting the plunger to the bottom of the S-Monovette® with the other hand **3b**. Loosen the tourniquet when collecting blood with the first S-Monovette®. Each additional S-Monovette® can be used for collection with the aspiration or vacuum technique, see also the instructions for use for S-Monovette® <https://www.sarstedt.com/en/download/instructions-for-use/>, Diagnostics / Venous blood / Monovettes / "SARSTEDT S-Monovette® Blood collection system".
- 4 After completing blood collection, first rotate the S-Monovette® a **¼ turn anticlockwise (I)** and then pull it out of the S-Monovette® safety needle/needle **(II) (disconnect) 4a**. The S-Monovette® safety needle/needle remains in the vein during blood collection and when changing the blood collection tubes. In the case of multiple collections – as described above – subsequent S-Monovettes are connected to the S-Monovette® safety needle/needle and further samples are taken. After collecting the required amount of blood, disconnect the last S-Monovette® from the S-Monovette® safety needle/needle and withdraw the S-Monovette® safety needle/needle from the vein **4b**.
- 5 **Safety needle:** Fit and activate the needle guard of the safety needle using the one-hand technique, either on a stable surface **5a** or with your index finger pressing against the lower end of the guard **5b**. The needle engages in the needle guard with a palpable and audible 'click'. **The safety mechanism is designed to be activated using the one-hand technique.**
- 6 Immediately dispose of the S-Monovette® needle or the safely locked safety needle in a suitable sharps disposal container **6**.

Handling: See handling video for the safety needle: www.sarstedt.com/en/download/videos/ Diagnostics / Venous blood / Monovettes

S-Monovette® safety needle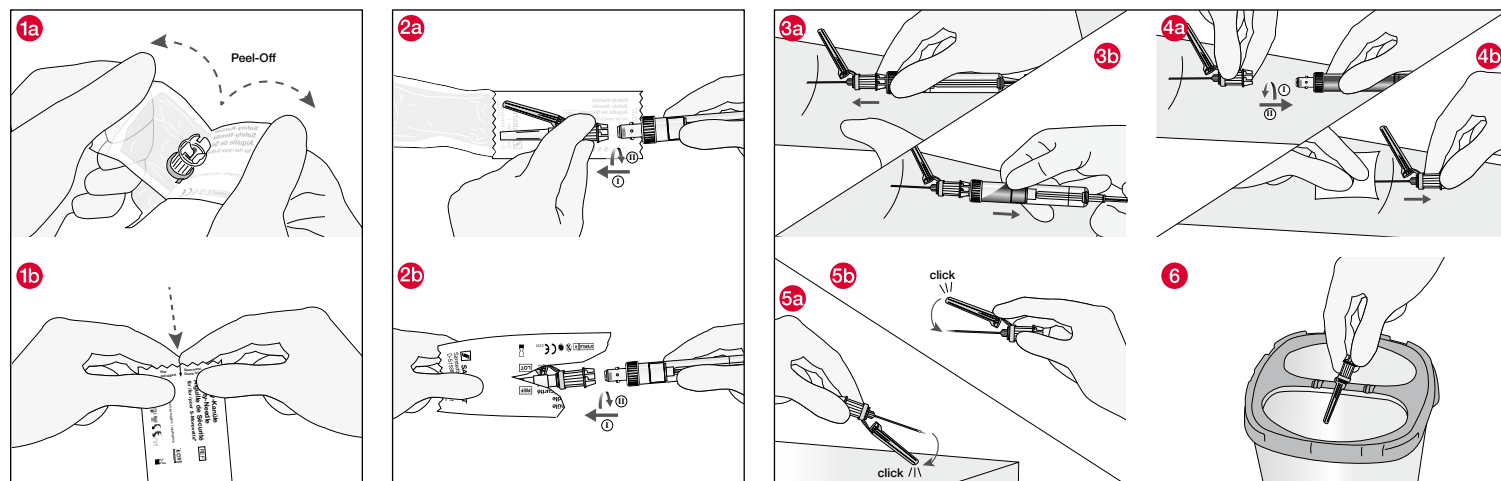

S-Monovette® needle

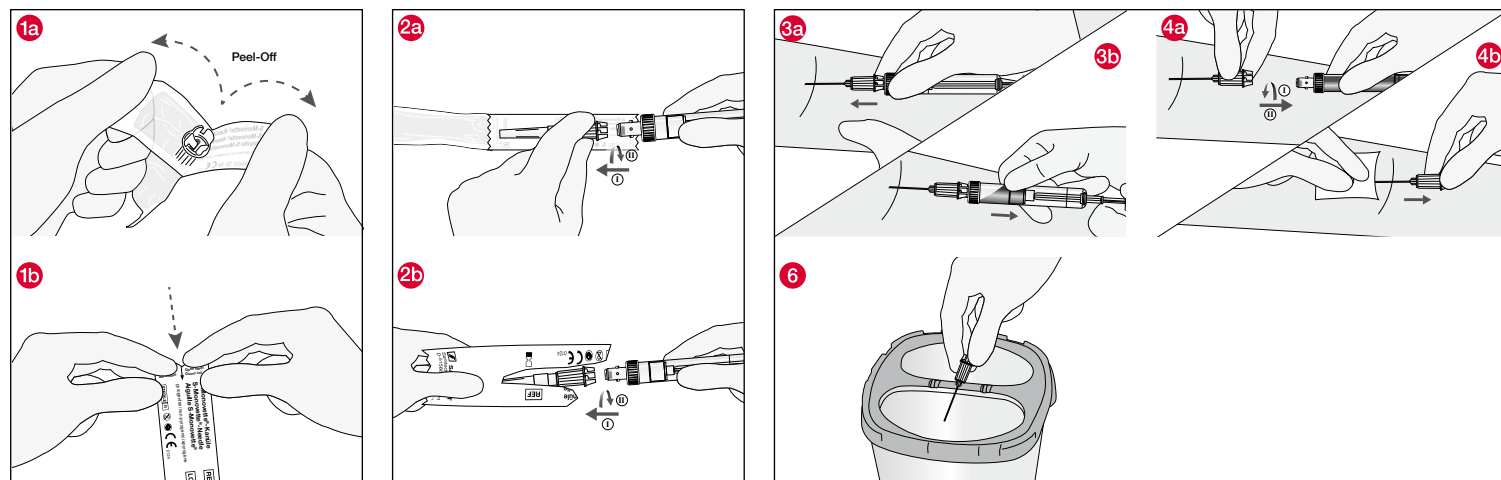Storage

Store the S-Monovette® safety needle/needle at room temperature.

NOTE: Do not expose to direct sunlight. Exceeding the maximum recommended storage temperature can impair the quality of the S-Monovette® safety needle/needle.

Product-specific standards and guidelines in the respective valid version

CLSI* GP41 Procedures for the Collection of Diagnostic Blood Specimens by Venipuncture.

*CLSI (Clinical and Laboratory Standards Institute)

WHO guidelines on drawing blood: best practice in phlebotomy

DIN 13097-4: Hypodermic needles - Part 4: Point geometry, requirements and testing.

DIN EN ISO 6009: Hypodermic needles for single use - Colour coding for identification.

DIN EN 10088-1: Stainless steels - Part 1: List of stainless steels.

DIN EN ISO 9626: Stainless steel needle tubing for the manufacture of medical devices - Requirements and test methods.

Key for symbols and labels:

	Medical device		Use by
	Article number		Follow the instructions for use
	Batch number		Country of manufacture
	CE mark		Keep away from sunlight
	Manufacturer		Store in a dry place
	Store at room temperature		Do not use if package is damaged
	Radiation sterilisation		Non-pyrogenic
	If reused: Risk of contamination		Single sterile barrier system
	Do not resterilise		Single sterile barrier system with outer protective packaging

Date of issue: 04/24

Technical modifications reserved.

All serious incidents relating to the product must be reported to the manufacturer and the competent national authority.

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