

- 1511 Proctoscope Diamond™
- 1521 CETRO® Proctoscope Medium Basic
- 1522 CETRO® Proctoscope Medium
- 1522-L CETRO® Proctoscope Medium Lume
- 1524 CETRO® Proctoscope Large
- 1531 CETRO® Proctoscope Small
- 1541 Proctoscope Select™
- 1542 CETRO® Anoscope
- 1551 CETRO® Tube Proctoscope
- 1561 CETRO® Rectoscope
- 1561-1 CETRO® Rectoscope+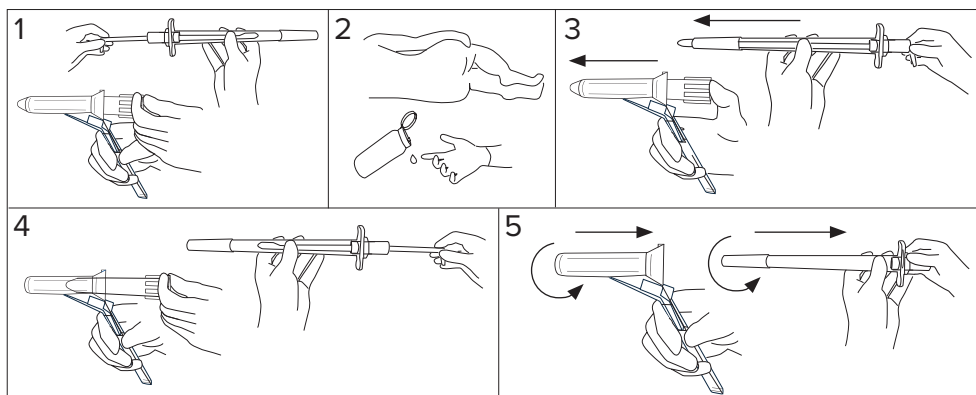

sv en no da fi de fr nl pl et lv lt it el pt es cs
bg ga hr mt ro sk sl hu ar sr

40-1500-240510

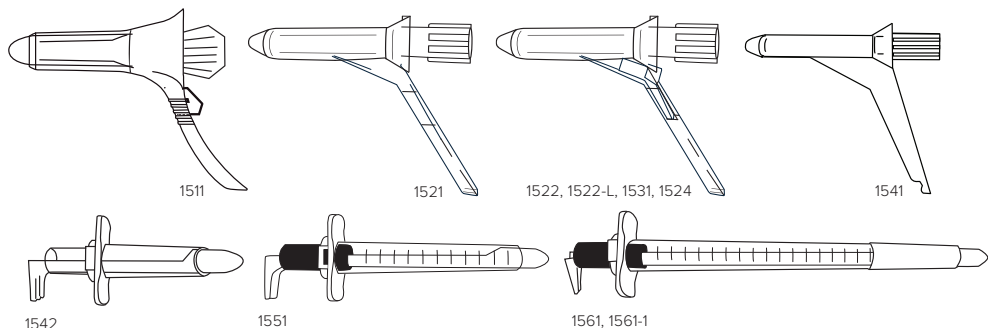CETRO MEDICAL

Cetro Medical AB, Nitgatan 11, SE-333 33 Smålandsstenar, Sweden, +46 371 330 30, info@cetromedical.se

1511 Proctoscope Diamond™, 1521 CETRO® Proctoscope Medium Basic, 1522 CETRO® Proctoscope Medium, 1522-L CETRO® Proctoscope Medium Lume, 1524 CETRO® Proctoscope Large, 1531 CETRO® Proctoscope Small, 1541 Proctoscope Select™, 1542 CETRO® Anoscope, 1551 CETRO® Tube Proctoscope, 1561 CETRO® Rectoscope, 1561-1 CETRO® Rectoscope+

SV

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en

Intended Purpose: The product is a single use, disposable device to be used by healthcare professionals during proctological examinations of rectum and anus.
Intended user: Healthcare professional, typically a proctologist or other professional working within proctology.

1. Take the disposable instrument out of its packaging. Inspect it to ensure it has no damage. Test the functionality outside the patient first.
2. The patient should lie on their side. Use a drop of lubricating gel on the finger and examine the anal opening, anal canal, and lower part of the rectum.
3. Insert the proctoscope into the rectum.
4. Remove the obturator.
5. Inspect the lower rectum and the entire anal canal by rotating and withdrawing the proctoscope.

The medical device should be disposed of as per respective hospital policy.
All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A) Consult instructions B) Catalogue number C) Batch code D) Medical device E) Keep away from sunlight F) Manufacturer G) Use-by date H) Do not use if package is damaged and consult instructions for use I) Keep dry J) Does not contain or presence of natural rubber latex K) Do not re-use L) Not sterile M) Complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

1511 Proctoscope Diamond™, 1521 CETRO® Proctoscope Medium Basic, 1522 CETRO® Proctoscope Medium, 1522-L CETRO® Proctoscope Medium Lume, 1524 CETRO® Proctoscope Large, 1531 CETRO® Proctoscope Small, 1541 Proctoscope Select™, 1542 CETRO® Anoscope, 1551 CETRO® Tube Proctoscope, 1561 CETRO® Rectoscope, 1561-1 CETRO® Rectoscope+

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