
CytoFixx Pump Spray

Instructions for Use


CytoFixx Pump Spray NHB-0100-01A / NHB-0101-01A / NHB-0200-01A

- 3 **EN** Instructions for Use
- 3 **FR** Instructions d'utilisation
- 4 **DE** Gebrauchsanweisung
- 4 **NL** Gebruiksaanwijzing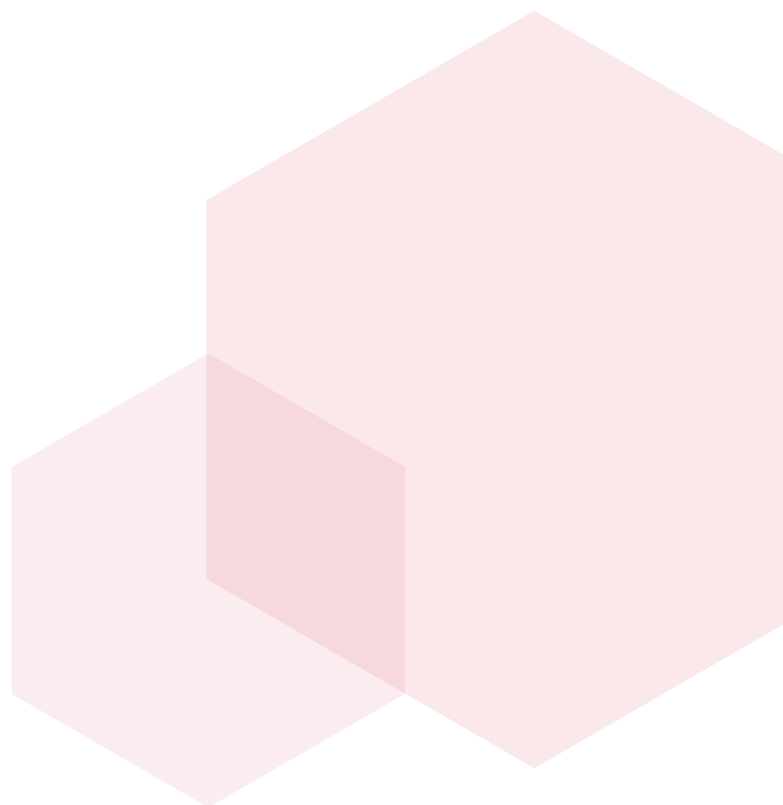

Components (Wt%)

Ethanol	≥ 90
Methanol	1 - <5

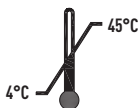Emergo Europe
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

CellPath Ltd
80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK
T: +44 (0)1686 611333
E: sales@cellpath.com
cellpath.com

CytoFixx Pump Spray NHB-0100-01A / NHB-0101-01A / NHB-0200-01A

EN

An alcohol-based solution intended to be used as a fixative for a cytology specimen obtained directly from a patient, or processing of a clinical specimen, for subsequent preservation, transportation, and/or in vitro diagnostic analysis.

Ready to use solution.

For laboratory use.

Keep in a cool, dry, well ventilated area. Keep containers tightly closed. Store in correctly labelled containers.

Please see expiry on product label.

Instructions for Use

- 1 Shake well before use.
- 2 Place slide on a horizontal surface.
- 3 Apply fixative immediately after smearing the cell specimen on the microscope slide, to avoid air drying.
- 4 If using the spray bottle, hold bottle 15cm away from the specimen when spraying.
- 5 Ensure specimen is adequately covered.
- 6 Pour off any excess fixative after a few minutes.
- 7 Replace cap after use.
- 8 Allow fixative to dry before mailing. Around 10-20 minutes from application.
- 9 Please ensure the first step of the staining protocol is a 50% ETOH rinse to remove the fixative prior to staining.
- 10 Dispose of empty bottle in accordance with local or government guidelines.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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CellPath Ltd, 80 Mochdre Enterprise Park, Newtown, Powys, SY16 4LE, UK