

Instruction for Use

CE0123

[Product Name]

Sterile Vaginal Speculum for Single Use

[Intended Use]

Sterile Vaginal Speculum for Single Use for vaginal expansion and gynecological examination of clinical use.

[Product Model]

SF-CK001, SF-CK002, SF-CK003,
SF-CK004, SF-CK005, SF-CK006

[REF NO]

VS0017, VS0018, VS0019, VS0027, VS0028, VS0029, VS0037, VS0038, VS0039, VS0047, VS0048, VS0049, VS0057, VS0057P, VS0058, VS0058P, VS0059, VS0059P, VS0056, VS0056P, VS0055, VS0065, VS0066, VS0067, VS0068, VS0069

[Intended User]

This product should be used by professional medical personnel after reading the instruction for Use.

[Intended Patient]

Patients who need clinical gynecological examination.

[Contraindications]

Not found.

[Warnings]

- Must be used within the specified validity period, packaging must be very tight and not damaged;
- Damp, mildew, leaks, damage as well as prohibit the use of the product failure;
- Can only be used once, can not be reused.
- If the packaging is damaged before use, it is forbidden to use it. During use, stop using it immediately when an accident occurs.

[Usage Methods]

First open the package, insert the front end of the speculum into the vagina, and then use the adjusting rod to adjust the speculum to the suitable inspection position, and it should be destroyed after use.

[Disposal of Waste]

Products after use should be destroyed and disposed of according to local/national regulations. It is strictly forbidden to discard it randomly.

[Expiry]

Valid period of product is 5 years.

[Storage and Transport Conditions]

Product should be stored in non-corrosive gases, no more than 80% relative humidity, clean, well-ventilated environment, transport should be to prevent the damage, stress, direct sunlight and rain and snow shower dip.

The product should be protected from heavy pressure, direct sunlight, rain and snow during transportation.

[Sterilization]

EO sterility

[Symbol Description]



CE Mark



Manufacturer



Date of manufacture



Batch code,



Consult instructions for use



Do not use if package is damaged



Do not re-sterilize



Keep dry



Medical Device mark
EU authorized representative



Use-by date



Do not re-use



Caution



Sterilized using ethylene oxide



Keep away from sunlight,

[Manufacturer]

Name: Changzhou Shunfeng Plastic Co., Ltd.
Address: No. 129, West Lane, Shunbei Village Committee, Shunbei Country, Zhenglu Town, Tianning District, 213116, Changzhou City, Jiangsu Province, PEOPLE'S REPUBLIC OF CHINA
Tel.: 0519-88902385
Fax: 0519-88909276
E-mail: sales@czsf.com

[EU Authorized representative]

Name: CMC Medical Devices & Drugs S.L.
Address: C/Horacio Lengo N° 18 CP 29006, Málaga-Spain
Tel.: +34951214054
Fax: +34952330100
E-mail: info@cmcmmedicaldevices.com

[Production Date]

Refer to the package label.

[Version effective date]

2024.11.01

[Revision No.]

F/1