
Mounting Media

SEA-1300-00A, SEA-1304-00A, SEA-1600-00A, SEA-1604-00A,
SEA-1900-00A, SEA-1904-00A, SEA-9000-00A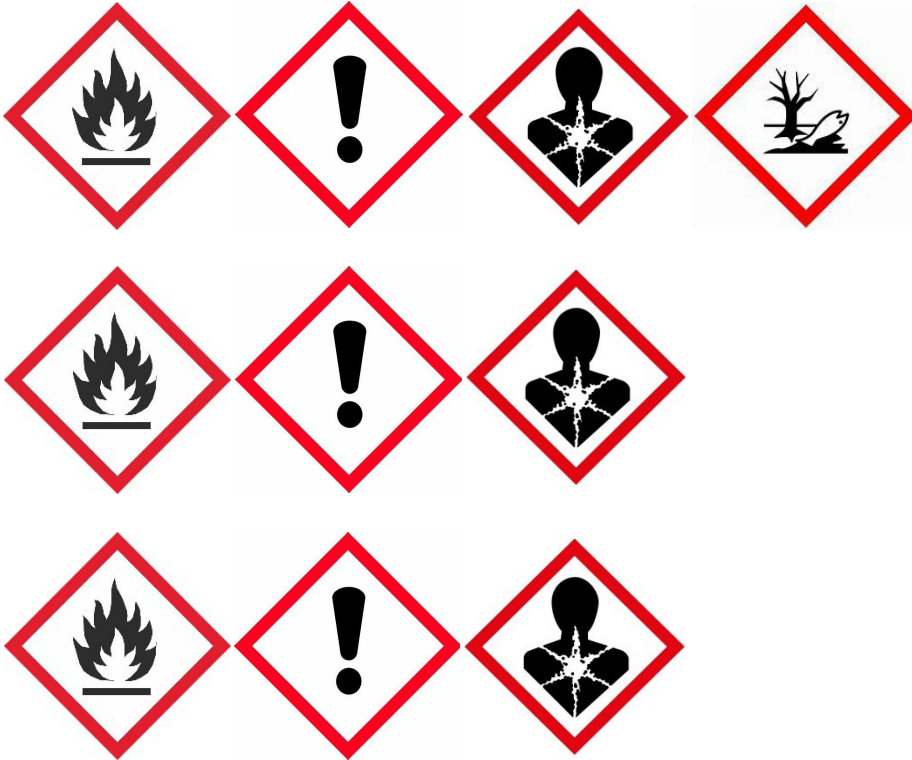

Expert XTF:

XTF Clearing Agent, 1- Dodecanethiol,
Isopar E, 2- Ethylhexyl Methacrylate,
Isobutyl Methacrylate

DPX:

Xylene, Ethel Benzene, Toluene

Expert:

Xylene, Ethel Benzene, Toluene

EMERGO EUROPE
Westervoortsedijk 60,
6827 AT Arnhem
The Netherlands

Issue Date: AUG 2023, 00.994.088 V3

CellPath Ltd

80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK

T: +44 (0)1686 611333

E: sales@cellpath.com

cellpath.com

Mounting Media

**SEA-1300-00A, SEA-1304-00A, SEA-1600-00A, SEA-1604-00A,
SEA-1900-00A, SEA-1904-00A, SEA-9000-00A**

A substance intended to be used to create a seal between a microscope slide, clinical specimen and/or coverslip to preserve the specimen and enhance image quality during microscopic examination.

Ready to use solution.

For laboratory use.

Keep in a cool, dry, well-ventilated area.

Keep containers tightly closed.

Store in correctly labelled containers.

Please see expiry on product label.

Clearing Agent Compatibility: DPX: Xylene; ExPert: Xylene;
ExPert XTF: XTF Clearing Agent.

Application – For manual or automated use.

Instructions for Use

Important: Leave to settle for 8 hours prior to use.

1. Use only in a well-ventilated area.
2. Slides should be well drained of clearing agent prior to mounting.
3. Apply mountant and coverslip to slide.
4. Allow the slides to dry in a horizontal position for 1 hour prior to moving or viewing on a microscope.
5. Allow the mounted slides to dry for 48 hours (DPX and ExPert) and 72 hours (ExPert XTF) prior to filing.

NOTE: The use of excessive heat to dry the slides is not recommended.

6. Dispose of empty bottle in line with government guidelines.

NOTE: The coverslip and DPX, ExPert or ExPert XTF mounting medium can be removed by soaking a mounted slide in the clearing agent.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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