Papanicolaou OG6 Stains



RBA-2504-00A, RBA-4211-00A

Contains | Enthält | Contient | Bevat:
Methanol | Méthanol

Emergo Europe,
Westervoortsedijk 60,
6827 AT Arnhem,
The Netherlands

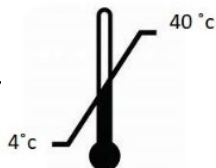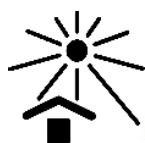Issue Date: MAY 2024 00.951.123 V5

CellPath Ltd

80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK

T: +44 (0)1686 611333

E: sales@cellpath.com

cellpath.com

Papanicolaou OG6 Stains

RBA-2504-00A, RBA-4211-00A

A chemical agent(s), dye(s) and/or solution(s) intended for the artificial colouration of specific target tissue structures, intra/extracellular elements and/or infectious microorganisms, in a clinical specimen, for their subsequent visualization, examination, identification and/or differentiation.

Ready to use solution. For professional use.

Components: Ethanol 75 - < 90%, Methanol 1 - < 5%.

Keep in a cool, dry, well-ventilated area. Keep containers tightly closed. Store in correctly labelled containers. Please see expiry on product label.

The Papanicolaou staining method consists of the use of a highly selective blue nuclear stain (**Harris Haematoxylin**), combined with a subtle polychromic counterstain (**EA50**) which differentiates cyanophil cells from eosinophil cells, and a secondary counter solution (**OG6**) which stains keratinized cells.

Expected Staining Results:

- Nuclei: Blue/purple
- Cyanophil cytoplasm: Blue/green
- Eosinophil cytoplasm: Pink
- Keratinized cytoplasm: From pink to orange

Troubleshooting:

Staining baths containing OG6 counter stain should be replaced regularly to ensure high quality staining. The time between reagent changes will vary from lab to lab depending on workload, staining protocol and reagent carry over. When cleaning baths with bleach, ensure they are thoroughly rinsed and scrubbed with clean water to prevent any residual bleach being left behind.

NOTE: All users should validate any stains against the lab's own protocols however for those unsure we recommend the use of Bancroft's (*Bancroft et al, 8th edn, ISBN 978-0702068645*)

Dispose in line with government guidelines.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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