



Microvette® 100/200 User Guide


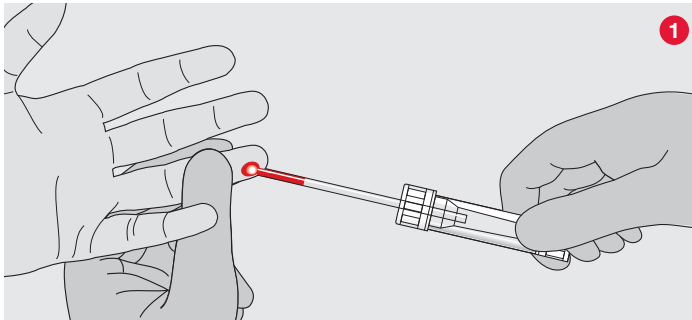


1. Collect blood with the assembled End-to-End 100 µl or 200 µl capillary. For collection, best results are achieved if the Microvette® is held in horizontal or slightly inclined position.



During blood collection, hold the capillary 3 to 5 mm away from the puncture site.

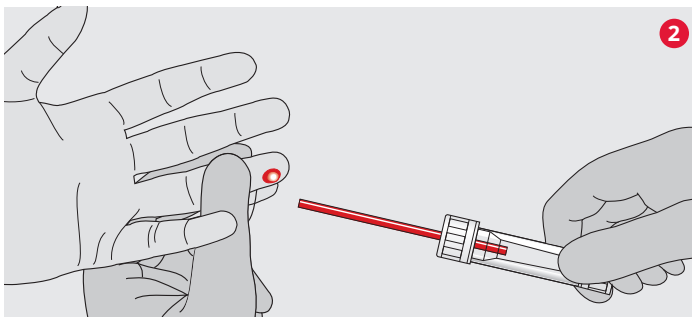

2. Collection is complete when the End-to-End capillary is entirely filled with blood.

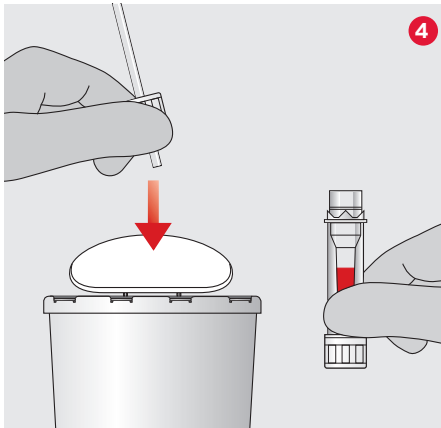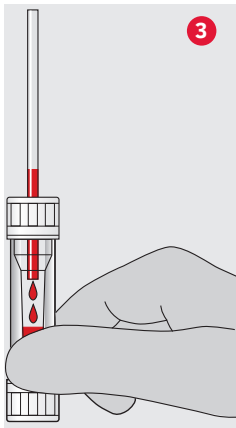

3. Hold the tube upright to allow the blood to flow from the capillary into the Microvette®.
4. Turn the cap to remove and discard the preassembled capillary as one unit.

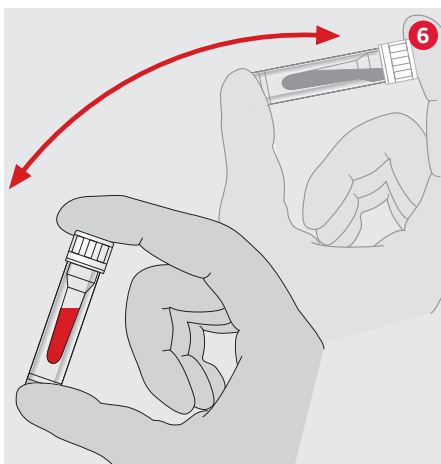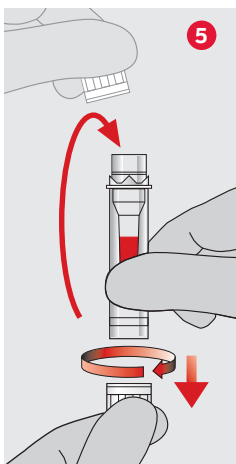


5. Remove the cap from the base and seal the Microvette® ('click' position).
6. Mix sample thoroughly by inverting the Microvette®.

**All patient blood specimens
should be treated with standard
precautions.**

Wear gloves!

Microvette® 300/500 User Guide

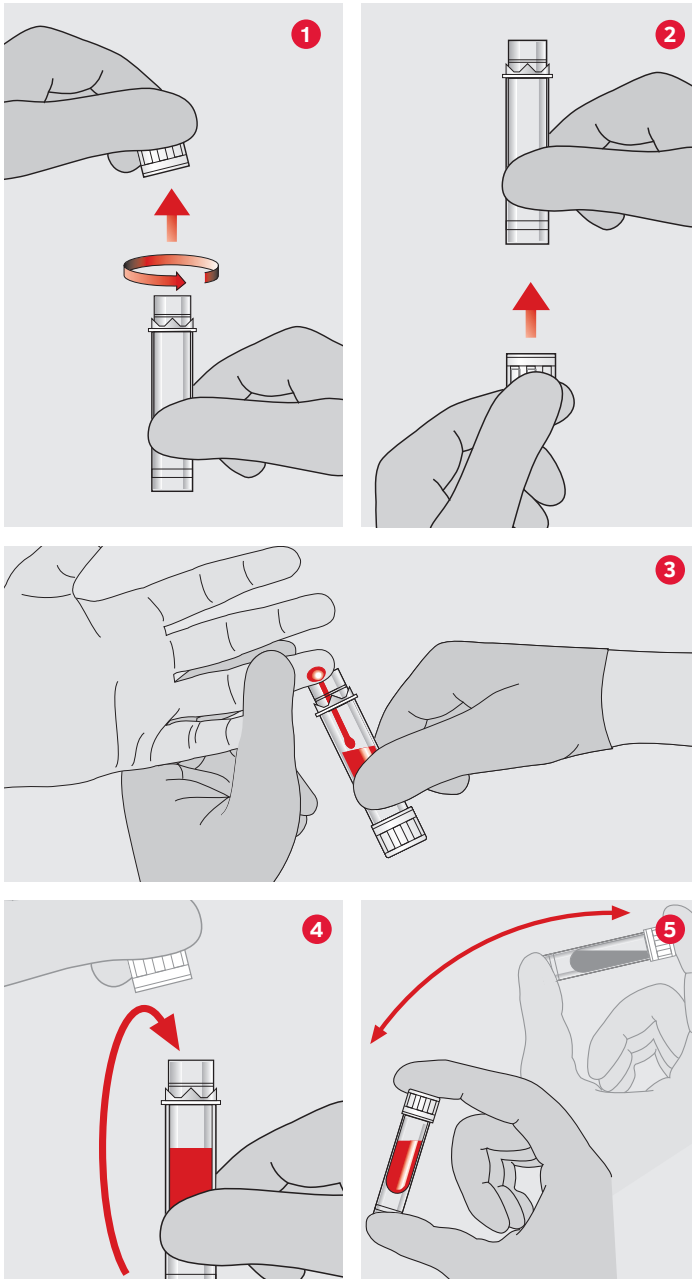

1. Turn the cap to remove it from the Microvette®.
2. Attach the cap to the base of the Microvette®.
3. Collect blood by gravity-flow using any part of the collection rim.
4. Remove the cap from the base and seal the Microvette® ('click' position).
5. Mix sample thoroughly by inverting the Microvette®.

**All patient blood specimens
should be treated with standard
precautions.**

Wear gloves!