



# Multi-Adapter with Blunt Cannula

## Instructions for Use


1. For use with ports with septum membrane. Follow manufacturer instructions and facility policy to prepare the arterial line port for specimen collection, including but not limited to discard volume removal and sampling port cleansing.
2. Tear package open at arrow and aseptically remove the Multi-Adapter with Blunt Cannula.

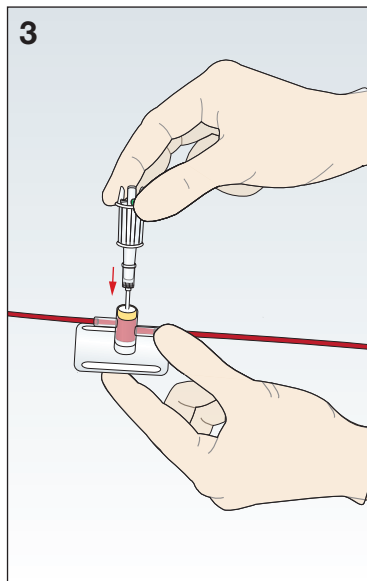


3. Push the blunt cannula end of the Multi-Adapter into the sampling port. Be sure to anchor the sampling port throughout the sampling process to avoid unwanted movement.

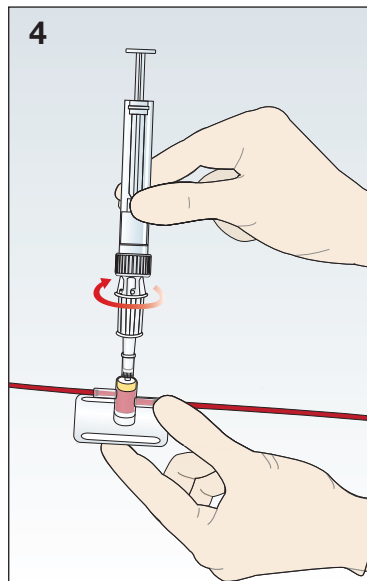


4. Connect S-Monovette® tubes to Multi-Adapter by twisting slightly clockwise.

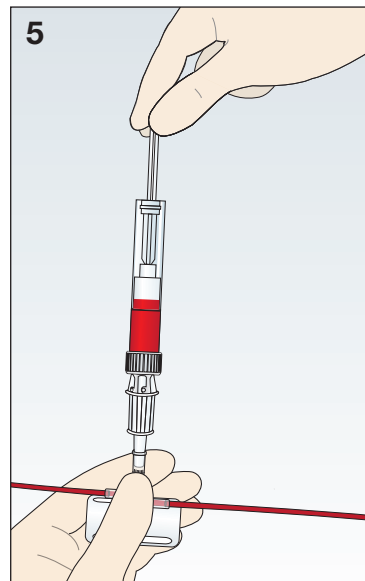

5. Hold Multi-Adapter with one hand and collect blood as needed by pulling back the piston rod.

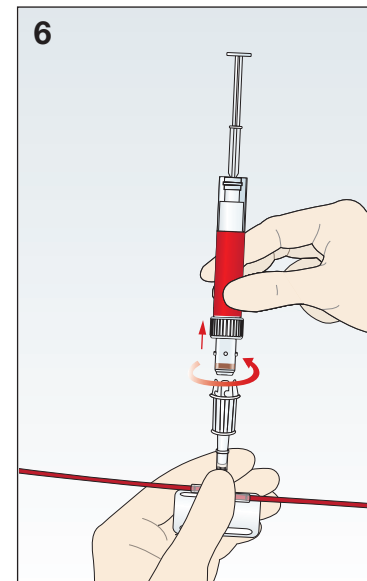

6. Disconnect filled S-Monovette® tubes from the Multi-Adapter by twisting slightly counter-clockwise.

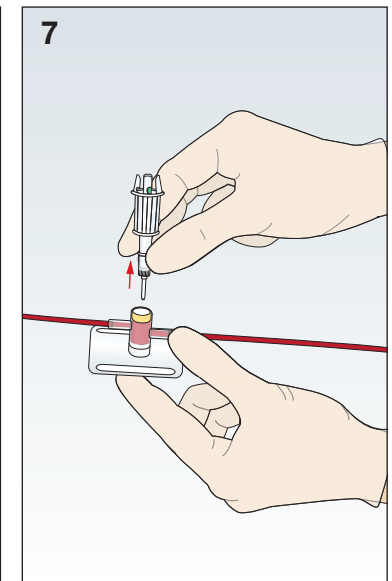

7. After the last S-Monovette® is disconnected, remove the Multi-Adapter by grasping the blunt cannula portion and pulling straight out of the sampling port.

8. Immediately dispose the Multi-Adapter with Blunt Cannula into an approved sharps container according to facility policy.
9. Visually ensure the sampling port membrane of the arterial line reseals properly before leaving the patient.